INSTRUCTIONS FOR USE

CUSTOM COMPOSITE RESIN

EN

1.PRODUCT DESCRIPTION

Custom Composite Resin is designed for LCD or DLP based 3D printers. The formulation is suitable for devices with wavelengths of 405 nm and 385 nm.

2.INTENDED USE

Custom Composite Resin is intended for the manufacturing of 3D printed temporary or permanent crowns and bridges. The product is classified as a biocompatible according to ISO 10993.

3.CONTRAINDICTION

Custom Composite Resin should not be used for any purposes other than the additive manufacturing for the produce of temporary or permanent crowns and bridges. Do not use in case of allergic reaction to any of the ingredients (contains methacrylate monomers and oligomers).

4.PROCESSING

Pay attention to work as clean as possible. Dirty equipment can cause defects in the printed part and damage the material tray.

4.1 BEFORE PRINTING

It is advised to use nitrile gloves when handling the Custom Composite Resin. Custom Composite Resin bottles should be well shaken before use and fill enough material into the tray. Remove any occurring bubbles with a clean spatula. Make sure the processing temperature adjusted up to 23°C to 30°C.

4.2 AFTER PRINTING

After the build process is finished remove the building platform from the machine. Release the printed parts with a cutter knife or spatula from the build platform.

4.3.CLEANING

Clean the printed parts for three minutes in ethanol (>90%) or isopropanol to remove any excess resin, using an ultrasonic bath or comparable wash-system until you get matt surface. The total cleaning time in ethanol/isopropanol should not take longer than 5 minutes.

4.4.POST-CURING

After cleaning of the parts, place the printed parts in an adequate light curing unit and cure at least 20 minutes. (200 W, 315-400 NM Light Curing Unit)

4.5.FINISHING

Remove the support structures with a cutter knife. You can finished the printed parts with a Professional dental tools. Fort he last step strain the resin and pour it back into the bottle.

5.STORAGE CONDITIONS

Custom Composite Resin should be stored dry (at 15 °C - 30 °C) and protected from light in the original packaging. Do not use after the expiry date.

6.DISPOSAL

Disposal of liquid resin must be carried out in accordance with the official regulations. Do not empty into drains. Custom Composite Resin in its cured form is not environmentally harmful.

7.HAZARD STATEMENTS

H302 + H312 – Harmful if swallowed or in contact with skin
H315 – Causes skin irritation
H319 – Causes serious eye irritation

8.PRECAUTIONARY STATEMENTS

P102 Keep out of reach of children.
P103 Read label before use.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P301+P330+P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do - continue rinsing.
P410+P412 Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122°F.
P501 Dispose of contents/container in accordance with local regulations.

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